

Why should I use a different number of needles?

DermaFocus Microneedling Pen has single-use cartridges that are available in a variety of needle quantities per head. Generally, fewer needles mean there is more pressure per needle, causing more trauma to the skin, making them particularly useful for severe signs of ageing or post-inflammatory marks. More needles create less pressure on the needle, so is better for comfortability or for larger areas such as the body.



12
Treating post-inflammatory marks, face



24
General use on face, neck, décolletage & scalp



36
For use on the body



Dr Kam Lally

“The DermaFocus Microneedling Pen has significantly enhanced the efficiency of our treatment process. The quick-change cartridge heads can be replaced within seconds, offering invaluable convenience on busy clinic days. Its lightweight, ergonomic design ensures exceptional comfort and control, even during extended treatment sessions.”

What is Microneedling?

Microneedling is a minimally invasive treatment, which involves puncturing the skin with very fine, sterile needles. This technique causes epidermal micro-channels, stimulating the body's natural wound healing response which can increase collagen production and allow topical products to penetrate more deeply into the skin.

What can microneedling help treat?

- Wrinkles
- Fine Lines
- Post-Blemish Marks
- Sagging Skin
- Uneven skin tone
- Dark Spots
- Thinning Hair



About the pen

DermaFocus Microneedling Pen has been developed to achieve incredible results on it's own or in combination with other treatments such as exosomes.

Microneedling Pen has anti-backflow technology, is totally wireless with a battery life of 2 hours per charge, and comes with 2 batteries. It supports 9, 12, 24, 36, 42, nano, 3D and 5D needle tips for desired treatments.



What needle length should I use?

The DermaFocus Microneedling Pen has variable needle lengths, from 0.25mm to 2.5mm. You can change the needle length easily mid-treatment by twisting the top of the pen until the arrow points towards the desired length.



1.5mm - 2.5mm

Wrinkles
Scarring
Stretch marks
Body



0.5mm - 1mm

Dark Spots
Fine Lines



0.25mm - 0.5mm

Scalp

How to use:

- | | |
|-----------------------------|--|
| 1 CLEAN THE SKIN | 4 APPLY THE GLIDING PRODUCT
(Purasomes) |
| 2 DISINFECT THE AREA | 5 START THE TREATMENT
following the microneedling technique: horizontal, vertical and diagonal lines |
| 3 SET THE PEN | 6 MORE PASSES WHERE PATIENT NEEDS
(Fine lines, scars, etc) |

Why should I use different speeds?

The DermaFocus Microneedling Pen features five adjustable speed settings (4,000-12,000 RPM), allowing you to tailor each treatment to the treatment area, skin condition and patient comfort. Higher speeds provide faster coverage for larger areas, while lower speeds offer greater precision and control when treating smaller or more delicate areas. The pen automatically starts at the highest speed and can be adjusted by pressing the power button to cycle through the five settings. Red is the highest speed, whilst green is the lowest.

